

ORTHOSAVE® COMP and ORTHOSAVE® DVT COMPRESSION THERAPIES

PATIENT USER GUIDE: KNEE WITH FOOT DVT

The OrthoSave® Comp provides controlled iceless compression therapy for the reduction of swelling and pain. In addition, the OrthoSave® DVT offers mobile, mechanical blood clot prevention. With certain conditions where a physician prescribes heat or contrast therapy, OrthoSave® Comp also delivers great outcomes.

YOUR DAILY PRESCRIPTION *(default prescription in parentheses below)*

Use for _____ (30) minutes with 1 hour off between sessions _____ (3-5) times per day.

1 WHAT'S IN THE BOX?

- ☒ OrthoSave® Comp
- ☒ Power cord
- ☒ Cold & compression wrap
- ☒ Distilled water
- ☒ OrthoSave® DVT
- ☒ Umbilical hose
- ☒ DVT wraps *(if applicable)*
- ☒ UPS return label

*Note: The machine color may be beige or dark grey

2 INITIAL SET-UP

ORTHOSAVE® COMP



STEP 1
Plug in power cord



STEP 2
Connect umbilical hose lines to **A** COMPRESSION, **B** WATER IN and **C** WATER OUT

ORTHOSAVE® DVT



STEP 3
Connect DVT wraps to **D** FOOT LEFT and **E** FOOT RIGHT

COMPLETED SET UP



3 STARTING THERAPY


STEP 1 — Apply all wraps prescribed by your physician.



STEP 2
Flip the switch **on** at the back of the machine.



STEP 4
Press button to start therapy.

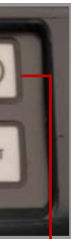

STEP 3
Hold button down until screen **turns on**.

4 STOPPING THERAPY



STEP 1
Hold button down until screen **turns off**.



STEP 2
*Flip the switch **off** at the back of the machine.
*Important: Do not switch off in the back before turning off the front of the unit

5

APPLICATION OF KNEE COLD COMPRESSION & DVT WRAPS

- STEP 1

Place DVT wraps around both feet, strapping them around the foot and then around the ankle with hoses going towards toes.
- STEP 2

Connect umbilical hose to cold compression wrap.
- STEP 3

Place the middle of knee wrap over knee cap and strap around leg.



COMPLETED KNEE SET UP

WATCH HOW TO SETUP THE ORTHOSAVE®

Scan this code on your smartphone to watch a video demonstrating steps 2-5.

6

TROUBLESHOOTING

MACHINE / SYMPTOM	POSSIBLE SOLUTION(S)
<div>ORTHOSAVE® DVT</div> <div>The OrthoSave® is beeping, but <i>shows no error code</i> on the display screen.</div>	<div>a. Remove DVT unit by pressing black plastic release button on the corner of the unit.</div> <div>b. Shut DVT unit off then back on to reset.</div> <div>c. Check connections and make sure they are plugged into the DVT ports.</div> <div>d. Check for pinched wrap air lines.</div> <div>e. Make sure DVT wraps are placed on the outer side of the calf with the clear tubes facing the outer side of leg/ankle.</div>
<div>ORTHOSAVE® COMP</div> <div>The OrthoSave® is beeping and <i>shows an error code</i> on the display screen. Fault codes will appear.</div>	<div>a. Fault code "2060": Low liquid in reservoir or hose not completely connected to wrap.</div> <div><div>▪ Shut comp unit off then turn back on to reset. Check line connections, if not connected you will hear/feel snap.</div><div>▪ How to refill the liquid reservoir when low:</div><div><div>1. Start the OrthoSave® Comp.</div><div>2. Remove cap from the top of the machine and begin refilling the machine with distilled water while the unit is running until the reservoir is full.</div><div>3. Once fill line is full, screw cap back to the top of the OrthoSave® Comp.</div></div></div> <div>b. Call your Sales Service Rep for any additional questions.</div>

7

RETURNING THE ORTHOSAVE®

STEP 1

Place the UPS return label on the box, covering any previous shipping labels.

STEP 2

Disconnect all hoses from the OrthoSave® Comp.

STEP 3

Throw away all wraps and pads in a trash can.

STEP 4

Place the machine, power cord, and umbilical hose in the box for return.
(Note the separate compartments for the OrthoSave® machine, power cord and umbilical hose)

STEP 5

Tape the box up so the machine is secure.

STEP 6

Connect with your Sales Service Rep and have them schedule UPS to pickup the equipment.

WATCH HOW TO RETURN THE ORTHOSAVE®

Scan this code with your smartphone to watch a video demonstrating on how to safely pack and return the OrthoSave®.